


# Micromedex<sup>®</sup> Clinical knowledge suite

## Nursing Pocket Reference Card

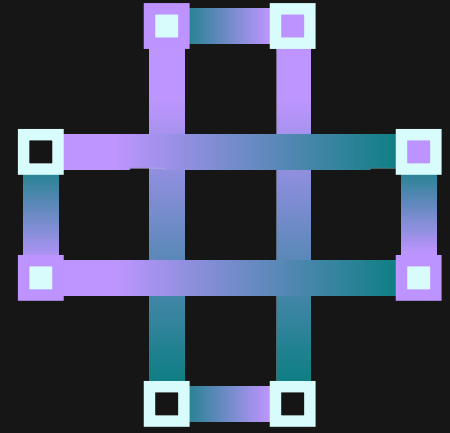

### ACLS / PALS

Select **Calculators** (or **Other Tools**) on the toolbar and locate **ACLS / PALS** below Dosing Tools. An **ACLS / PALS** treatment calculator is available when weight is entered.

### Adverse effects

Search for a drug. In the monograph, select **Adverse Effect** located below **Medication Safety**.

### Clinical checklist

From disease **Quick Answer** or **In Depth**, select Clinical Checklist from **Related Results** shown on the right side of the page. Example: Search atrial fibrillation and view what treatment should always be / never be performed.

### Drug consults

Under **Resources**, the **Drug Consult** link returns an alphabetical list of useful consults such as: Acetaminophen / Ibuprofen Pediatric Dosing, Immunization Guidelines Timing and Spacing, and Urine Drug Screen False Positives.

### Crushed

Searching for the term "**Crushed**" produces an alphabetical list of capsules / tablets that should not be crushed.

### Cytotoxic drug extravasation therapy (also Non-Cytotoxic)

Searching for these terms delivers Drug Consult results for **Cytotoxic Drug Extravasation Therapy** (or Non-Cytotoxic Drug Extravasation Therapy).

### Enteral feeding

Searching for "**Enteral Feeding**" returns the Enteral Feeding Tubes and Medication Delivery Drug Consult.

### Searching for dialyzability

Searches using a drug name typically go directly to **Quick Answers**. Click the **Pharmacokinetics** section to find **Dialyzability** included in **Excretion**.

### Mobile app passwords

Passwords are found under **Mobile Application Access** section accessed from your organization's Micromedex home page.

© Copyright Merative US L.P. 2023. Merative 100 Phoenix Drive Ann Arbor, Michigan 48108

Produced in the United States of America December 2023

Merative, the Merative logo, and Micromedex are trademarks of Merative in the United States, other countries or both. All other company or product names are registered trademarks or trademarks of their respective companies.

This document is current as of the initial date of publication and may be changed by Merative at any time. Not all offerings are available in every country in which Merative operates.

The information in this document is provided "as is" without any warranty, express or implied, including without any warranties of merchantability, fitness for a particular purpose and any warranty or condition of non-infringement. Merative products are warranted according to the terms and conditions of the agreements under which they are provided.

The client is responsible for ensuring compliance with all applicable laws and regulations applicable to it. Merative does not provide legal advice or represent or warrant that its services or products will ensure that the client is in compliance with any law or regulation. The performance data and client examples cited are presented for illustrative purposes only. Actual performance results may vary depending on the specific configurations and operating conditions. It is the user's responsibility to evaluate and verify the operation of any other products or programs with Merative product and programs.

Statement of Good Security Practices: IT system security involves protecting systems and information through prevention, detection and response to improper access from within and outside your enterprise. Improper access can result in information being altered, destroyed, misappropriated or misused or can result in damage to or misuse of your systems, including for use in attacks on others. No IT system or product should be considered completely secure and no single product, service or security measure can be completely effective in preventing improper use or access. Merative systems, products and services are designed to be part of a lawful, comprehensive security approach, which will necessarily involve additional operational procedures, and may require other systems, products or services to be most effective. Merative does not warrant that any systems, product or services are immune from, or will make your enterprise immune from, the malicious or illegal conduct of any party.